


TIPS BACTODENT SAMPLE

Sample collecting and processing procedure

Diagnostic test for periodonitis

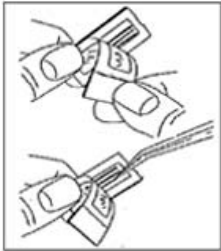


Bag with material sampling

- Contains a box with a transparent plastic cover with five sterile "tips".

Sampling process

- Pull back the top edge of the drilled area without removing the entire plastic cover so as to prevent the "tips" from falling.
- Remove makeup or lipstick, if they are used, before taking the sample.
- Before sampling, keep gums and teeth dry using sterile gauze. Take no mechanical dental cleaning prior to sampling.
- With sterile tweezers, take a "tip" by the end with the black line, and insert its other end into the bottom of the periodontal pocket.
- Repeat this operation with the five "tips" for four other periodontal pockets. Keep them for 20-30 seconds.
- Remove and put them with the end down into the plastic tube labeled "BACTOdent".
- Once all the "tips" are inside the tube, cover and label it with patient's name or references.

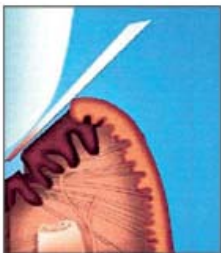

Sample shipment

- Place the sample in the padded envelope with the EUGENOMIC address.
- Include the Application form, completed properly, signed and labeled.
- Remember to indicate the SENDER data on the back of the envelope.
- Submit by mail.

